(EN)

# VULNAMIN® SPRAY

Adjuvant in the healing process

## DO NOT USE FOR INDICATIONS OR APPLICATION THAT ARE NOT SPECIFICALLY NOTED IN THE INSTRUCTIONS FOR USE

### INTENDED USE

This is a medical device made in compliance with the Directive 93/42/EEC MDD. It is intended to promote the restoration of a moist microenvironment in especially dry wounds, also while aiding the cleansing and granulation phase in uncleaned skin wounds.

### DESCRIPTION

Fluid gel containing amino acids that form collagen, in a liquid gel formula, aimed at promoting the formation of granulation tissue and tissue regeneration. It also facilitates wound cleansing.

### COMPOSITION

PURIFIED WATER, SODIUM HYALURONATE, GLYCINE, L-PROLINE, L-LEUCINE, LYSINE HYDROCHLORIDE, METHYL 4-HYDROXYBENZOATE SODIUM SALT, PROPYL 4-HYDROXYBENZOATE SODIUM SALT, PROPYLENE GLYCOLE, TETRASODIUM EDTA.

### PACKAGING

Glass bottle with dispenser - 30 ml spray.

### MODE OF ACTION

Vulnamin® spray protects skin integrity and can be used on lesions in the early stages, preventing lesion evolution to later stages. Vulnamin® spray aids detersion and formation of granulation tissue, and therefore regeneration of the wound area, in various types of skin lesions. Vulnamin® spray also reduces inflammatory infiltrate and stimulates fibroblasts to produce collagen.

### INSTRUCTIONS FOR USE

The versatile formulation of Vulnamin® spray allows use of the product to be used in difficult-to-reach areas that are particularly painful to the touch. Applying onto skin that is not yet ulcerated but which has the early signs of ulcer development helps to maintain the protective microenvironment on skin integrity. In the treatment of skin ulcers of various origins, with delayed healing, lesions with average or low exudate, whether cleansed or colonised, flat or with cavity, Vulnamin® spray facilitates cleansing and aids recovery of tissue repair. Do not use on open infections. In colonised wounds, use with a secondary anti-bacterial dressing.

### PRECAUTIONS FOR USE

Remove any foreign matter from the wound by spraying saline solution outwards from the centre of the wound. Wash hands thoroughly before medicating the wound. After thoroughly cleansing the area to be treated, pat the wound gently with gauze, to remove any excess blood. If the wound requires no further intervention, apply Vulnamin® spray as described in the indications for use, after cleaning the wound.

### CONTRAINDICATIONS

Not indicated in the event of known hypersensitivity to the ingredients.

There are no known cases of overdose.

There are no known side effects.

There are no known interactions with drugs and medicinal substances. Although there are no known secondary effects during pregnancy and lactation, in these cases the product should only be used after consulting a doctor.

Vulnamin® spray is a well-tolerated product; occasionally a burning sensation may be felt, due to the product's hypertonicity, which is a characteristic of the cleaning efficacy. This sensation will usually resolve itself spontaneously.

### WARNINGS

- The expiration date refers to product properly stored in a sealed package
- Product intended for external use
- Do not use after the expiry date
- Do not sterilise
- Do not use if the package is open or damaged
- Do not use in the event of known hypersensitivity to the ingredients
- Drying materials should not be used together with Vulnamin® spray
- Keep out of the reach of children
- In the event of adverse reactions, discontinue treatment and consult your doctor

### DOSAGE AND ADMINISTRATION

Spray Vulnamin® spray onto the skin affected by preulcerous lesions once or twice daily.

Vulnamin® spray is also indicated for ulcerous lesions of the skin without necrosis and infection (red phase). The interval between changes in dressing mainly depends on the state of the wound. Dressings must be changed in line with wound exudation; we recommend that dressing be changed on alternate days or, at most every three days. Wash and cleanse the wound and spray a suitable amount of product, using the most appropriate secondary dressing in line with the state of the wound.

### DISPOSAL

Please dispose of carefully.

### STORAGE

Vulnamin® spray must be stored at a temperature between 8 °C and 30 °C.

Do not store in a refrigerator. Store in a cool, dry place, away from moisture and sources of heat.

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